

# THE SCITON EDGE

## SCITONPRO.COM

Sciton professional website provided free of charge:

- Digital practice marketing assets
- Available 24/7

## SMART START™ GUIDE

5 steps to operating a thriving aesthetic practice:

- STEP 1: Get launch ready
- STEP 2: Become an expert
- STEP 3: Prepare your staff and office
- STEP 4: Know your patient
- STEP 5: Develop your marketing plan

## SUCCESS BUILDER™

Access to experienced third-party professionals and tools to support your practice growth.

- Clinical education
- Marketing tools
- Business development



### SPECIFICATIONS: CLEAR SUITE 1064 nm

Wavelength	1064 nm Nd:YAG
Fluence	Up to 400 J/cm <sup>2</sup>
Pulse Width	Up to 200 ms
Repetition Rate	Up to 20 Hz
Spot Size	ClearSilk: 7 mm ClearV: 2-6 mm ClearHair: 30 mm x 30 mm (scanned area)
Skin Cooling	ClearV: Adjustable, 5 °C to 25 °C
Contact Chiller	250 W thermoelectric chiller, continuous contact sapphire plate

### SPECIFICATIONS: BBL

Wavelength	420 – 1400 nm
Filters	420 nm, 515 nm, 560 nm, 590 nm, 640 nm, 695 nm, 800 nm
Output	Up to 30 J/cm <sup>2</sup>
Pulse Width	Up to 200 ms
Repetition Rate	Up to 1 pulse-per-second
Cooling Method	Continuous thermoelectric sapphire plates
Skin Cooling	Adjustable from 0 - 30 °C in 1 °C increments
Spot Size	15 x 45 mm
Finesse Adapters	15 x 15 mm square, 11 mm & 7 mm round

# Clear Suite

## THE FAMILY OF BRANDS

ClearV™

ClearSilk™

ClearHair™

AND

BBL™  
BroadBand Light

 SCITON®  
BECAUSE RESULTS MATTER




©2020 Sciton, Inc. All rights reserved. Sciton and HALO are registered trademarks and JOULE, ClearV, ClearSilk, ClearHair, BroadBand Light (BBL), Finesse Adapters, Smart Filters, Forever Young BBL, Resurfacing, LAPG and SVI are trademarks of Sciton, Inc.

www.Sciton.com •    2600-029-34 Rev. B

 SCITON®  
BECAUSE RESULTS MATTER

# Clear Suite

Sciton brings together the powerful combination of laser and light to treat a myriad of indications which further elevates the JOULE™ platform as the most customizable laser and light platform in the industry. Sciton is excited to provide you with a comprehensive vascular solution for your practice that ensures a continuous revenue stream and a great Return on Investment.

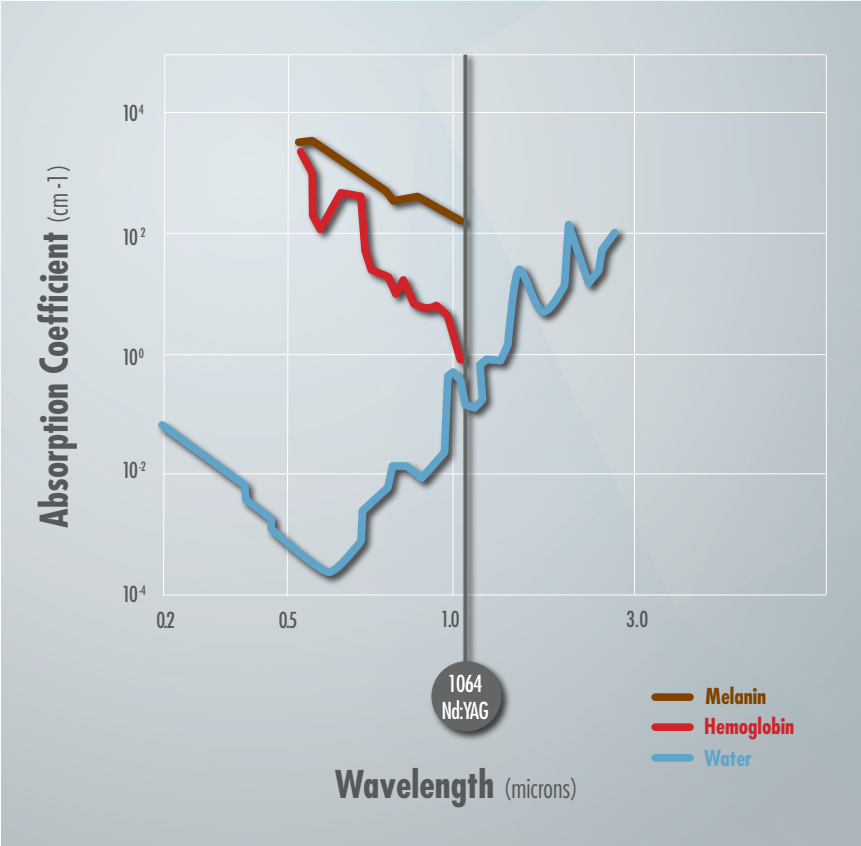

Clear Suite and BroadBand Light (BBL™)  
Product Comparison Chart

Applications		ClearV™ 1064 nm	ClearSilk™ 1064 nm	ClearHair™ 1064 nm	BBL™
SKIN QUALITY	Acne				•
	Acne Scars				•
	Pigmented Lesions		•		•
	Scar Revision		•		
	Skin Maintenance		•		•
	Wrinkles		•		
	Fine lines		•		•
VASCULAR	Vascular Lesions	•	•		•
	Telangiectasia	•			•
	Reticular/Spider Veins	•			
	Diffuse Redness		•		
HAIR & SPECIALTIES	Permanent Hair Reduction			•	•
	Onychomycosis (ClearToe)		•		
	Warts	•			
OTHER	Skin Types	All	All	All	I-V
	Anesthetic (numbing) Required	None	None	None	None
	Days of Downtime	None	None	None	None
	All Seasons	•	•	•	•



Sherrif F. Ibrahim, MD

“

As a dermatologist I appreciate having an entire arsenal of modalities in one platform to treat multiple skin conditions. The Clear Suite in combination with BroadBand Light™ offers me an unmatched capability to treat vascular lesions and improve the signs of sun damaged skin.

The Clear Suite 1064 Nd:YAG applications – ClearV™, ClearSilk™, and ClearHair™ – combined with BroadBand Light (BBL™) offer you the unique capability to treat vascular lesions, skin revitalization, and hair from a single platform.

## VEINS:

# ClearV™



### Treat the Veins You Can and Can't See!

ClearV™ is the preferred choice for vascular treatments. It effectively and safely treats visible vessels, broken capillaries, spider veins, vascular lesions, and skin discoloration. Undesirable visible micro vessels will be gently eliminated. Resistant unwanted subsurface vessels will be heated by the laser energy and will fade over time and become less noticeable.

### The Preferred Choice for Vascular Treatments

ClearV™ effectively and safely treats visible vessels, broken capillaries, spider veins, vascular lesions, and skin discoloration. Visible vascular structures will be gently eliminated. Unwanted vessels will be heated by the laser energy and become less noticeable.

### Effective Treatment for All Skin Types

The ClearV treatment is the treatment of choice for all skin types (Fitzpatrick I – VI). Patients have virtually no downtime and may resume daily activities immediately following treatment.

## Indications

- Facial telangiectasias
- Leg veins
- Periorbital veins
- Reticular veins
- Spider veins
- Port wine stains
- Rosacea
- Reticular veins
- Skin tags
- Warts



Antonio Voegeli-Campo, MD


“

Sciton's ClearV™ is the new gold standard for vascular treatments! It's unique Subsurface Vein Illumination allows me to clearly see veins below the surface and even treat feeder veins I can't see for a more complete and precise treatment. Coupled with integrated cooling and adjustable spot sizes, ClearV makes the treatment much more comfortable for the patient and easier for the provider.

## Technology

### Subsurface Vein Illumination (SVI™)

SVI technology is specially designed to give the provider enhanced visibility to see subsurface and feeder veins to treat more thoroughly and effectively. Push button controls on the handpiece offer the ability to change the lighting color, level of brightness, and aiming beam intensity.

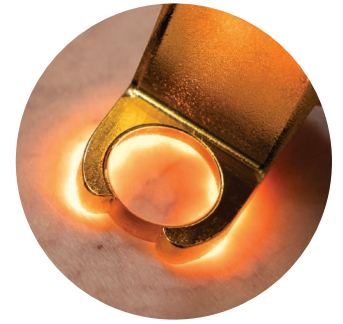

**Subsurface Vein Illumination (SVI™)**  
Enhanced visibility of feeder veins to treat more completely and effectively

### Adjustable Integrated Cooling

Sapphire tip provides optimal cooling pre, during and post as well as bulk cooling for patient comfort. The adjustable cooling range of 5-25 °C is ideal for treating a variety of vascular conditions safely, efficaciously, and efficiently.

### VascuZoom

VascuZoom enables quick and easy spot size changes directly on the handpiece. This enables the user to switch effortlessly between 2-6 mm spot sizes.

## ClearV Benefits

- Subsurface Vein Illumination (SVI™)
- Integrated cooling
- All Fitzpatrick skin types
- No disposables
- Combine with ClearSilk™, HALO®, Resurfacing™ and BBL™ treatments to offer best in class treatments
- No patient downtime - treat all year long



ClearV™ | 3 months post 2 tx | courtesy of courtesy of Peter Castillo, MD



ClearV™ | 1 month post 1 tx | courtesy of courtesy of Peter Castillo, MD



SKIN + VASCULAR:



BroadBand Light (BBL™) is the world’s most powerful IPL device. It sets new standards in treating skin conditions associated with aging, active life styles, and sun damage. The innovative BBL module extends the versatility of the JOULE platform with one-of-a-kind dual lamp technology, interchangeable Smart Filters™, Finesse Adapters™, and CoolComfort™ technology.

Technology

BBL enables treatment of multiple skin conditions with various wavelengths seamlessly using change-on-the-fly Smart Filters.

Wavelength (nm)	INDICATIONS
420	Mild to moderate acne vulgaris
515	Pigmented Lesions, incl. Lentigos (age spots), Solar Lentigos (sun spots)
560	Vascular Lesions, such as Telangiectasias, and Rosacea
590	Port Wine Stains, Hemangioma, Vascular Malformations
590, 640 & 695	Hair Removal
590, 695 & 800	Skin Firming

Dual-Lamp Technology

BBL’s dual-lamp technology is uniquely designed with each lamp supplying half the energy during a pulse resulting in an extended lifetime of the BBL handpiece.

Now with CoolComfort™ Technology

A unique precision cooling system featuring the most powerful cooling capacity allowing the user to set and maintain cooling from 1-30 °C in 1 °C increments for fast, comfortable and uniform treatments.

Treatment Speed

BBL allows for faster treatments than other IPL devices. This is thanks to its large spot size (15x45 mm), the high repetition rate and the easily exchangeable snap-on Finesse Adapters™ as well Smart Filters™.

Uniform Beam & Square Wave Energy Delivery

Precisely engineered parabolic optics deliver energy to the treatment area uniformly, producing no hot or cold spots, which results in perfectly consistent treatments.

Application Driven Interface

Our intuitive interface provides step-by-step parameters for our most popular treatments allowing the user to quickly master the art of BBL treatments.

BBL Benefits

- Treat all demographics
- Combine with ClearV™, ClearSilk™, Resurfacing™, and HALO® to offer best in class treatments
- Fast, effective, versatile, and cost effective
- Predictable and consistent
- User friendly



FYBBL™ | 3 years post 5 tx | courtesy of Robyn Siperstein, MD, FAAD



BBL™ | 1 month post 2 tx | courtesy of Antonio Campo-Voegeli, MD

## SKIN + VASCULAR:

# ClearSilk™



### Address Minor Skin Imperfections and Boost Skin Revitalization with a Non-Ablative No Downtime Solution

The ClearSilk™ treatment works by deposition of heat into micro vessels. This heat increases the temperature of the blood that is flowing through the micro vessels, stimulating fibroblast activity and the body's natural repair process resulting in a decrease in fine lines and wrinkles. ClearSilk is the treatment of choice for all skin types (Fitzpatrick I – VI). Patients have no downtime and may resume daily activities immediately following treatment.

## Indications

- Fine lines and wrinkles
- Improved luminosity
- Pigmented lesion
- Acne scars
- Diffuse redness
- Skin revitalization
- Rosacea

## Technology

### Built-in Temperature Sensing

Easily visualize the skin temperature as you are treating with the built-in sensing gauge that indicates temperature ranges between 32-48 °C.

### Audible Feedback

The ClearSilk handpiece and JOULE system deliver audible alarms to provide real time feedback on the number of pulses delivered.



Antonio Voegeli-Campo, MD

“

*I have used 1064 Nd:YAG for years and I find the ClearSilk™ in combination with BBL™ to be a superior choice for treating rosacea. The combination boosts the effect on the control of flushing and inflammatory flare-ups. No other system offers me the versatility that my JOULE™ provides.*

### Visual Distance Guide

Enable user to maintain consistent treatment distance to target for uniform heating to ensure reliable results.

## ClearSilk Benefits

- All Fitzpatrick skin types
- Easily delegatable
- Zero downtime
- 4-6 treatments recommended to achieve optimal results
- No anesthetic or disposables required
- Comfortable with minimal patient discomfort
- Combine with other Sciton modalities to offer best-in-class treatments
- Non-ablative, and non-contact



ClearSilk™ | 3 weeks post 7 tx | courtesy of Peter Castillo, MD



ClearSilk™ | 4 tx | courtesy of Peter Castillo, MD



ClearSilk™ + FYBBL™ | 6 years of yearly routine tx, total of 8 tx  
courtesy of Antonio Campo Voegeli, MD





### A Safe and Effective Solution for Permanent Hair Reduction

The ClearHair™ 1064 nm Nd:YAG wavelength and ideal pulse duration are unmatched for permanent hair reduction. The 1064 nm wavelength has strong absorption in melanin, providing selective heating of hair follicles for excellent clinical results.


### Efficacious Treatment for All Skin Types

ClearHair is the treatment of choice for all skin types (Fitzpatrick I – VI). ClearHair 1064 nm wavelength and external high capacity cooling allows users to reduce hair permanently with ultimate safety and efficacy.

## Technology

### Large Area Pattern Generator (LAPG™) and Non-Sequential Scanning

ClearHair utilizes a LAPG high speed scanner and non-sequential energy delivery to achieve optimal clinical outcomes. The computer-guided scanner covers large areas rapidly and evenly on treated areas to maximize the time interval between adjacent spots, allowing heat to dissipate. This technology also minimizes subsurface heat build-up, providing additional comfort during treatment.

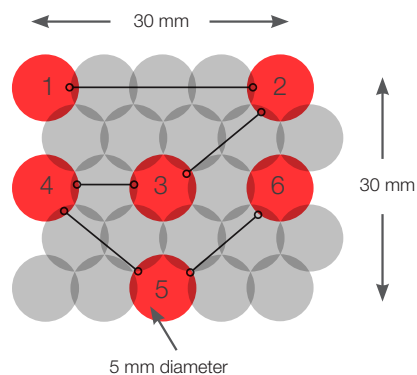

LAPG™ – Large Area Pattern Generator:  
Non-sequential scanner technology achieves optimum overlap resulting in a safe, uniform, and efficacious treatment

### High Speed Treatment

ClearHair covers large areas up to 4x faster than other systems, for example requiring only 200 scans to cover a large back.

### Integrated Continuous Cooling

ClearHair uses an external high capacity chiller to remove the high heat load when large areas are treated. Its 250 watt cooling capacity is 10x greater than built-in chiller handpieces and enables the highest level of safety and patient comfort.

## ClearHair Benefits

- Proven to be safe and efficacious for all skin types (Fitzpatrick I – VI) for permanent hair reduction.
- No downtime
- No consumables



ClearHair™ | 12 tx | courtesy of Lory Haney, RN



ClearHair™ | 40 days post 1 tx | courtesy of Terumi Liu, MD  
Area treated: lower lip and chin, demonstrated on left side